

Installation Instructions

1. Using the screws and washers circled in figure A, attach the base (1) of the cart to the support column (2). You will need the provided 5mm hex key for this.
2. Now attach the cable hook (3) to the support column (2) using the screws circled in figure B using a Phillips head screwdriver
3. Slide the lower shelf (4) along the groove on the front of support column (2) and tighten with the attached knobs. This can be adjusted according to your preference
4. Attach the main shelf (5) to the support column (2) using the screws in figure C with a 4mm hex key.
5. Attach the monitor support column (6) to the main shelf (5) using the screws and washers in figure D with a 5mm hex key.
6. **Two people required:** Once the cart has been constructed, lock the wheels, and mount the Prosidio L.E. Endoscopy Camera system carefully onto the VESA mount at the top of the monitor support column (6) using the provided screws using a 3mm hex key

